

About Us: Soaids is a trusted mobility and care solution brand dedicated to enhancing the lives of seniors, including those with mobility issues. With thoughtful design, quality materials, and a deep understanding of aging needs, our innovative solutions empower older adults to live more independently and confidently, whether at home, outdoors, or in care facilities.

At Soaids, we believe that growing older should not mean giving up independence. We design every product with empathy, dignity, and purpose—to support seniors in aging gracefully, securely, and confidently.

With our in-house R&D, quality manufacturing, and deep empathy for the elderly, we create solutions beyond products—they are partners in daily life.

Join Soaids in making life easier, safer, and more enjoyable.

✉ rd@suncare-medical.com

🌐 <http://www.suncare-medical.com>

Multi Shower Commode Chair with Wheels Installation Instructions Model: SC8188A



Setup & Operation




● Before using this product:

- Please read all instructions carefully.
- The commode chair features a shower chair, bedside commode, transport wheelchair, or over the toilet, making daily care easier.

Assembly Instructions

● Prepare the Components

- Remove all parts from the packaging and place them on a clean, flat surface.
- Make sure all components listed above are included.

Number	Picture	Name	Total quantity
A		Wheelchair Base	1
B		Backrest	1
C		Armrest	2

D		Cushion	1
E		Bucket	1

● Step 1: Attach the Backrest (Part B)

- Insert the backrest (Part B) into the slots at the top rear of the Wheelchair Base (Part A).
- Push down firmly until it clicks into place.
- Make sure both sides are aligned—this will make installation easier.

● Step 2: Install Armrest (Part C)

- Align the armrest with the position, press the button, and push it in.
- Lower the armrest and engage it into the backrest position.

● Step 3-4: Install Bucket (Part E) and Cushion (Part D)

- Place the bucket and mat on the sitting board.
- Adjust the height position of the pedal.
- Engage and test the brakes on the wheels.



Product Specs:

- Product Weight: 19.3 lbs
- Maximum Weight Capacity: 300 lbs



Cleaning & Maintenance

- Remove the bucket and clean with warm water and mild detergent.
- Wipe down the frame and seat with a damp cloth.
- Avoid harsh chemicals or abrasives.