

Before You Begin

Thank you for choosing Soaids 3-Wheel Rollator Walker. Please read this manual carefully before use to ensure safe and proper operation.

Warranty Information

This product is covered by a one-year limited warranty from the date of purchase. During the warranty period, if the product experiences issues not caused by misuse or accidental damage, we will provide free repair or replacement.

If you receive the product with missing parts or minor damage due to shipping, please contact us immediately. Our support team will respond and resolve any product-related concerns within 24 hours.

Product Name	Product Model	Purchase Date	User Name
Repair Remember record	Repair Content	Repair Date	Maintenance Personnel

⚠ Warning & Safety Precautions:

Do not use strong acids, solvents, or corrosive cleaners on any part of the rollator.

Do not hang bags or heavy items on the handles, as this may cause the walker to tip over.

Keep children away from the rollator to avoid tipping accidents or injury.

Do not use the walker if it shows signs of shipping damage or structural issues.

Exercise caution during installation.

When using outdoors, avoid heavy rain to prevent slipping.

3-Wheel Rollator Walker User Guide SC890L



Product Overview

Product Name: 3-Wheel Rollator Walker

Model: SC890L

Maximum Load Capacity: 260Lbs

Package Dimensions: 17"(L) x 13"(W) x 7 1/10"(H)

Overall Dimensions: 27 4/5"(L) x 26 3/5"(W) x


32 3/5" - 36 3/5"(H)



Key Features

1. Stable Y-shaped detachable frame design;
2. Ergonomic armrests made of durable nylon;
3. 8-inch solid PVC wheels (front & rear) for smooth, cushioned movement;
4. Butterfly-style ergonomic handles for comfortable grip;
5. Dual-function braking system: slow brake/parking brake;
6. Reflective safety strips for nighttime visibility;
7. One-step folding mechanism—folds in seconds, no tools required;
8. Large-capacity shopping bag made of premium PVC synthetic leather.

Component Identification

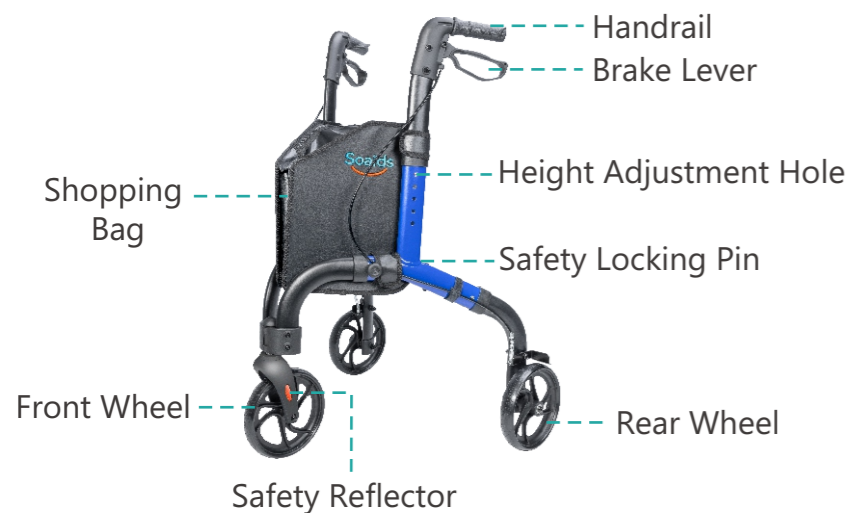

Package Contents

a	Main frame with handles assembly × 1
b	Front frame with front wheel assembly × 1
c	Left foot wheel tube × 1
d	Right foot wheel tube × 1
e	M6×35 external hexagonal screw × 2 M6 plum blossom knob × 2
f	Tool set × 1

Assembly Instructions

● Step 1: Install the Front Frame

Insert the left and right tubes of the front frame into the corresponding front tubes of the main frame.

Secure using the hex screws and star knobs.

Ensure the hex screw head fits fully into the hex-shaped slot.




● Step 2: Install the Rear Wheels & Adjust Handles

Insert the pre-assembled left and right rear caster tubes into the rear ends of the main frame.

Make sure the spring-loaded metal balls click into place.

Insert the handles and press the release button to adjust handles height as needed.

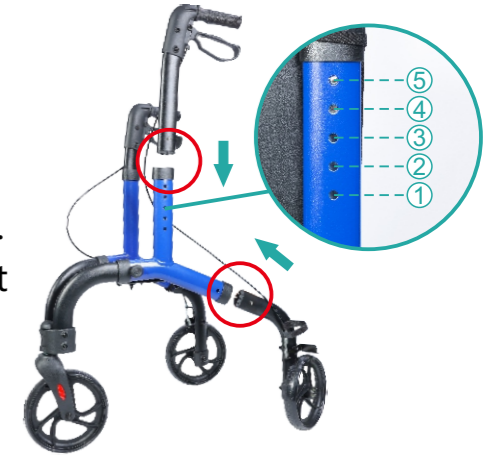

● Step 3: Install the Shopping Bag

Attach the shopping bag to the front frame tubes and the handle adjustment tubes of both sides.

Ensure all straps are secured.



● Step 4: Fold the Walker

Press the button shown in the illustration to fold the walker.

Folding requires only one press—quick and effortless.



Important Notes

When unfolding the walker, use the handle strap and avoid placing fingers near moving joints.
Make sure all parts are locked securely before use.